



NOBAZELLTUPF®

REF 208810

Product Description, Intended use, Application

The cellulose swab roll is provided with perforations, which allow an easy taking of the swabs. Each swab has a size of 8.0 cm x 10.0 cm.

The swabs are suited for the preparation of simple punctures with low risks, for example vein punctures for blood sampling. Prior to the above activity, the product must be sterilized using a validated procedure (ethylene oxide, radiation, moist heat at 121°C)

Composition

Cellulose

Note

Sterilize before use

Contraindications

The product should not be used in the case of a known allergy against the material.

Incident reporting

According to MDR (EU) 2017/745, if serious incidents occur in relation to the device, they must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Normative and Regulative Requirements, Common Standards

Medical Device according to MDD 93/42/EEC, MDR (EU) 2017/745.

The product does not contain dangerous toxic substances according to REACH.

Packaging

Primary packaging: PE bag

Secondary packaging: carton made of cellulose

Symbols used in labelling

Explanations at www.nobamed.com



Marking on all packaging levels with CE and according to DIN EN ISO 15223-1- and DIN EN ISO 20417

Storage and Transport

Dry and dustfree

Single use device

Reusing a single use medical device can lead to microbiological danger. Reprocessing, for reuse can decrease the product's performance significantly.

Disposal

According to locally applicable legal regulations and standards of infection prophylaxis.